


## Instructions For Use


# De Mayo Single Lever Clamp

Catalog Numbers	903			Status	APPROVED
Effective Date	May 11, 2020	Version	4	Page 1 of 2	

1. Position the clamp onto the O.R. table rail and with the lever in “unlock” position.

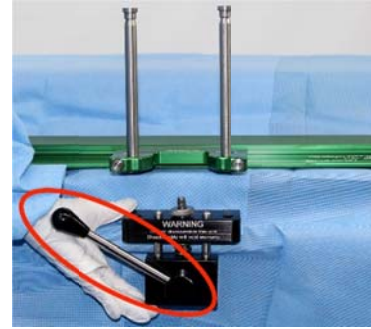

2. Rotate lever to “lock” position.

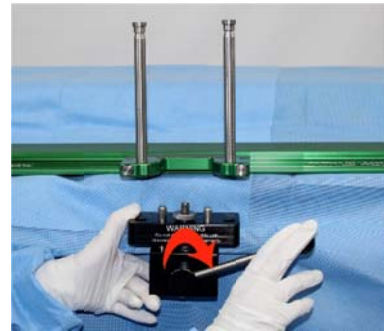


3. Squeeze jaws tight on side rail and tighten knurled knob clockwise all the way.



4. While holding clamp, rotate lever to “unlock”.




5. Turn knurled knob clockwise 1/4 to 1/2 rotation.

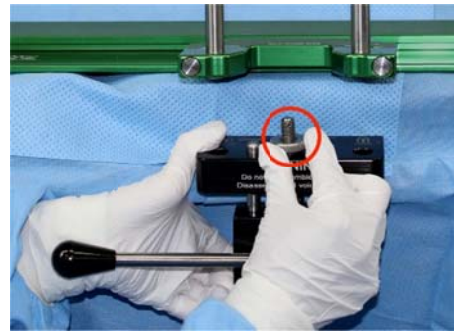

6. Rotate lever to “lock” position requiring only 2 finger pressure to achieve locking. Works like a vice grip and will “snap” into place.




7. Slide and guide pins into holes of the clamp.

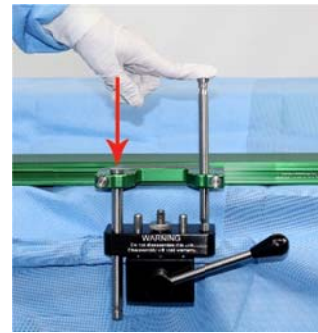

## Safety Test



1. Ensure clamp knob is tightened
2. Lever in the “locked” position to the right side of the clamp
3. Check that both guide pins are fully seated

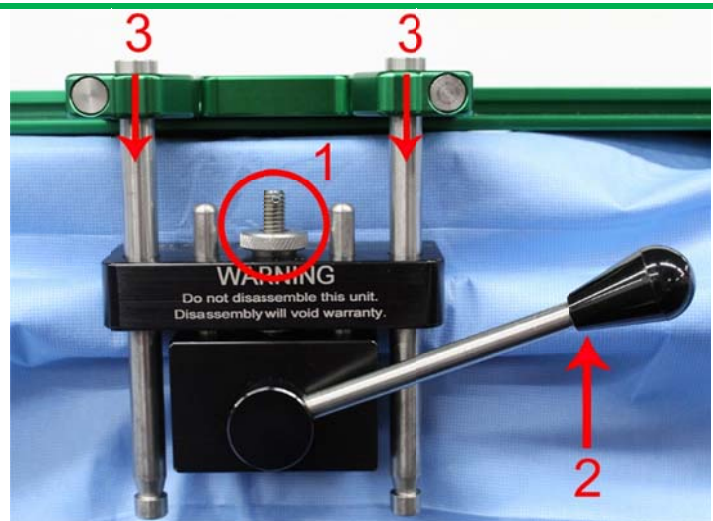

CE

imp® products are protected by patent & patent pending rights ~ go to [impatents.com](http://impatents.com)  
 © April 2020 Innovative Medical Products, Inc.  
 ALL RIGHTS RESERVED

Innovative Medical Products  
 87 Spring Lane  
 Plainville, CT 06062  
 P: 860-793-0391  
 F: 866-459-1805

Scan for additional documentation



TMP-IMP-0007 April 21, 2020 Rev 2