

## Info Sheet

# De Mayo D<sup>2</sup><sup>®</sup> Knee Positioner<sup>®</sup> Carriage Removal and Replacement

Catalog Numbers	<b>803-114, 803-114-S</b>		Status	<b>APPROVED</b>
Effective Date	<b>May 7, 2020</b>	Version	<b>2</b>	Page 1 of 2

## Carriage Removal

1. Unscrew the Thumbscrew with attached spring and remove the handle, then remove entire threaded rod with attached L Plate using your fingers.



2. Slide the carriage toward the "Loading End" of the Base Plate and slide the carriage around the curved end of the top plate.



3. Remove the E-Break from the carriage.



## Carriage Replacement

1. While holding the Teflon Pad, slide the carriage onto the top plate on "Loading End" starting on the handle side, the around the curved end of the Base Plate.



2. Using your fingers screw in the threaded rod with attached L Plate into the carriage until a boot ball fits into the carriage.



3. Replace the handle in the 3 o'clock position and screw in the Thumbscrew with attached spring until tight.



4. With the boot locked into place, pull the handle out and use the Thumbscrew to make any final adjustments and replace the E-Brake.



## Safety Test

1. Check that the handle locks the boot Ball and is in the 3 o'clock position
2. The E-Break is replaced



De Mayo D<sup>2</sup>® is a registered trademark of Innovative Medical Products, Inc.  
 imp® products are protected by patent & patent pending rights ~ go to [www.imppatents.com](http://www.imppatents.com)  
 © May 2020 Innovative Medical Products, Inc.  
 ALL RIGHTS RESERVED



**Innovative Medical Products**  
 87 Spring Lane  
 Plainville, CT 06062  
 P: 860-793-0391  
 F: 866-459-1805