



CAT# 803-114 or 803-114-S
De Mayo D² Knee Positioners
CLEANING PROCEDURE
With Optional Pin Base Removal

NOTE: ALL SOLUTIONS MUST BE COMPATIBLE WITH ALUMINIUM AND STAINLESS STEEL

Recommended Washer / Decontaminator Instructions:

- Remove carriage from positioner – Reference De Mayo D² Knee Positioner Carriage Instructions for use.
- Optional: Pin Base does not need to be removed from baseplate but if desired then remove from baseplate per:
 - De Mayo 1210-SP Base Disassembly and Assembly
- **NOTE: All carriage components including the handle, boot and baseplate should be controlled so that the same components all get reassembled together.**
- Soak the product in an enzyme solution (example: TRI-POWER ENZYMATIC CLEANER from UNITED BIOTECH www.united-biotech.net or similar). Follow the manufacturer's direction for dilution and soaking time. ***Validated by 3rd party for 15 minute soak time with Tri-Power**
- Put thru washer / decontaminator according to manufacturer's instructions with a **detergent up to a pH of 9.0**

NOTE: SELECT CYCLE THAT DOES NOT INCLUDE LUBRICATION

- Recommended Hand Cleaning Instructions:
- Pre-Soak the product / components in an enzyme solution. Follow the manufacturer's direction for dilution ratio and soaking time.
- Rinse the product in warm tap water.
- Wash the product with an instrument detergent **up to a pH of 9.0.**
- Rinse the product in warm tap water.
- Soak or wipe the product down with a hospital approved and / or EPA approved germicide according to instructions.
- Rinse the product in warm tap water.
- **DO NOT LUBRICATE THE KNEE POSITIONER PRIOR TO STERILIZATION OR IN THE OR.**
- Assemble carriage to baseplate.
- Assemble Pin Base CAT# 1210-SP to baseplate. Ensure original boot is placed with the assembled baseplate.
- Dry thoroughly and wrap.