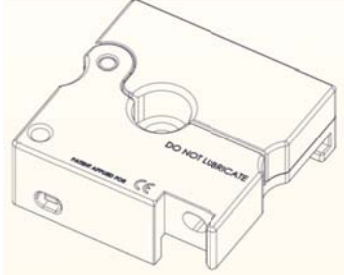
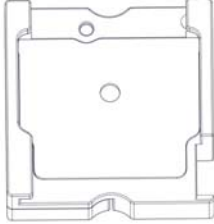
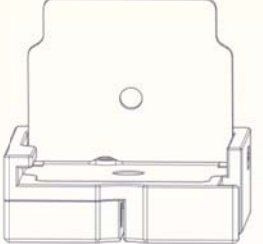
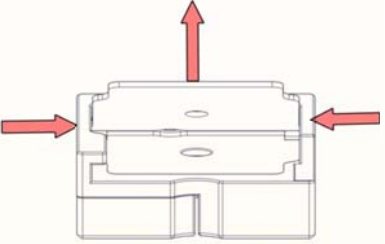
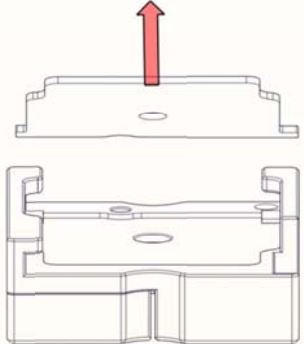






**CAT# Teflon Pad Removal and Re-assembly
For D2 & V2 De Mayo Knee Positioners (Rev 02-2019)
CAT# 114, 114-S, 803-114, 803-114-S**

Disassembly

#	Action	Visual
1.	Remove Carriage from Base plate	
2.	Flip the carriage over	
3.	Flip up the Teflon pad	
4.	Curl the Teflon pad up so that the tabs on the side are clear of the carriage	
5.	Lift out the old Teflon pad	

**CAT# Teflon Pad Removal and Re-assembly
For D2 & V2 De Mayo Knee Positioners (Rev 02-2019)
CAT# 114, 114-S, 803-114, 803-114-S**

Assembly

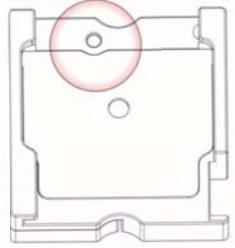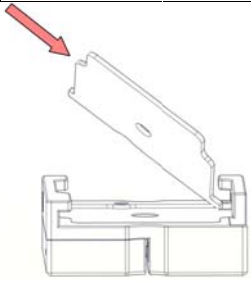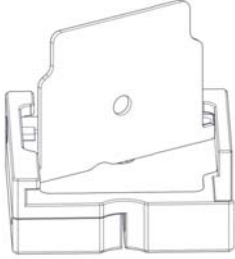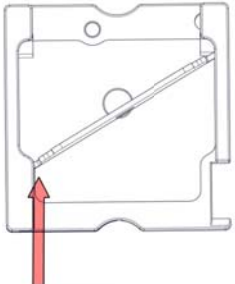1.	Ensure that the cut-out in the Teflon pad is around the pivot arm pin	
2.	Place the new Teflon pad Tab into one side of the carriage	
3.	Place the other tab into the carriage	
4.	Push on the tab to pop it into the hole	
5.	Seat the Teflon pad into the carriage. Make sure it is as flat as possible.	