



De Mayo Knee Positioner - Accessories
CLEANING PROCEDURE

NOTE: ALL SOLUTIONS MUST BE COMPATIBLE WITH ALUMINIUM AND STAINLESS STEEL

Recommended Washer / Decontaminator Instructions:

- Soak the product in an enzyme solution. Follow the manufacturer's direction for dilution and soaking time.
- Put thru washer / decontaminator according to manufacturer's instructions with a **detergent up to a PH of 9.0**

NOTE: SELECT CYCLE THAT DOES NOT INCLUDE LUBRICATION

- Recommended Hand Cleaning Instructions:
- Pre-Soak the product / components in an enzyme solution. Follow the manufacturer's direction for dilution ratio and soaking time.
- Rinse the product in warm tap water.
- Wash the product with an instrument detergent **up to a pH of 9.0.**
- Rinse the product in warm tap water.
- Soak or wipe the product down with a hospital approved and / or EPA approved germicide according to instructions.
- Rinse the product in warm tap water.
- Dry thoroughly and wrap.
- Sterilize per Sterilization Procedure.

Accessories	Part Numbers
Clamps	302, 903, 713-302, 713-717 or 114-CL
Boots	803-ABD or 803-CBD
Distractor	907