



De Mayo Knee Positioners STERILIZATION PROCEDURE

Recommended Sterilization Instructions

- Insure that all parts are thoroughly cleaned.
- Make sure that all moveable parts are loose and can move freely.
- **DO NOT PLACE POSITIONER IN A MILK BATH OR LUBRICATE.**
- If using a sterilization case follow Instructions For Use for the product.
- Double wrap in two disposable wraps. Use 48 inch x 48 inch wraps.
(approximately 122 cm x 122 cm).
- Run normal vacuum cycle for your institution
- **STEAM STERILIZATION ONLY- ALL OTHER STERILIZATION METHODS NOT VALIDATED**

MINIMUM PARAMETERS PRE-VAC STERILIZATION

Product Part Number	With Sterilization Case Case Part Number	Temperature Setting	Exposure Time	Dry Time
De Mayo Knee Positioner 803 & 803-S	706-S	270°F to 272°F 132°C to 134° C	4 minutes	20 minutes
De Mayo D ² Knee Positioner 803-114	706-S	270°F to 272°F 132°C to 134° C	4 minutes	30 minutes
De Mayo V ² Knee Positioner 114 & 114-S	706-S	270°F to 272°F 132°C to 134° C	4 minutes	30 minutes
De Mayo V ² E Extension Arm 114 - EXT	706-S	270°F to 272°F 132°C to 134° C	4 minutes	40 minutes
	Without Sterilization Case			
De Mayo Knee Positioner 803 & 803-S	Without Sterilization Case Double Wrap Only	270°F to 272°F 132°C to 134° C	4 minutes	20 minutes
De Mayo D ² Knee Positioner 803-114 & 803-114-S	Without Sterilization Case Double Wrap Only	270°F to 272°F 132°C to 134° C	4 minutes	20 minutes
De Mayo V ² Knee Positioner 114 & 114-S	Without Sterilization Case Double Wrap Only	270°F to 272°F 132°C to 134° C	4 minutes	20 minutes
De Mayo V ² E Extension Arm 114 - EXT	Without Sterilization Case Double Wrap Only	270°F to 272°F 132°C to 134° C	4 minutes	20 minutes
Accessories				
1 Clamp: 302, 903, 713-302, 713-717 or 114-CL 1 Boot: 803-ABD or 803-CBD 1 Distractor: 907	706-XS OR 706-M	270°F to 272°F 132°C to 134° C	4 minutes	20 minutes
1 Clamp: 302, 903, 713-302, 713-717 or 114-CL 1 Boot: 803-ABD or 803-CBD 1 Distractor: 907	No Case Wrap Only	270°F to 272°F 132°C to 134° C	4 minutes	20 minutes

CAUTION: POSITIONERS MUST BE COOL BEFORE APPLYING TO THE PATIENT.

Sterilization Parameters Certified by:

- Micro Test Laboratories (now Accuratus Lab Services)
- Accuratus Lab Services
- HIGHPOWER Validation Testing and Lab Services