

Instructions For Use

De Mayo Universal Distractor® with the De Mayo Knee Positioner® System

Catalog Numbers	907	Status	APPROVED
Effective Date	April 6, 2020	Version	3
		Page 1 of 2	

1. Once the carriage is locked in place, ensure the “E- Brake” is engaged as it provides extra locking pressure when using the Distractor
2. Place Patient Protective Pad® (CAT # 907-GP-10) on support bar.
3. Place support stem into appropriate distractor block slot and place padded support bar on the back of the patient’s thigh



4. Squeeze the large green handle to distract



5. Press small stainless steel lever to release the distraction



Safety Test

1. Ensure patient protective pad is on support bar.
2. Confirm support stem is in the appropriate distractor block



Scan for additional documentation

