

imp De Mayo V² E[®] Knee Positioner Instructions For Use ~ **STERILE Extension**

1. Locate the patient's gluteal fold near the break at the end of the table.



2. Apply a well leg holder to the non-operative leg.

3. Drape the patient,




lower the end of the table and remove the pad.




4. To position the Clamp:


Method 1: Find the end of the rail at the break and move the Clamp proximal approximately 7 inches.

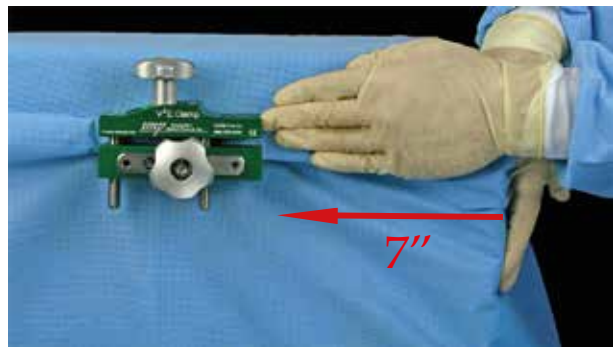


To lock the Clamp on the side rail, squeeze the jaws and tighten the top Knob.

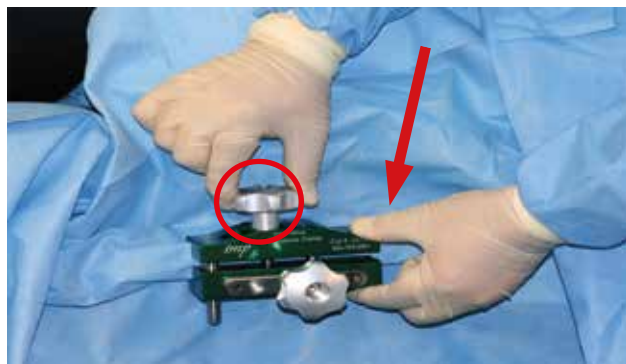

imp De Mayo V² E[®] Knee Positioner IFU ~ **STERILE Extension**

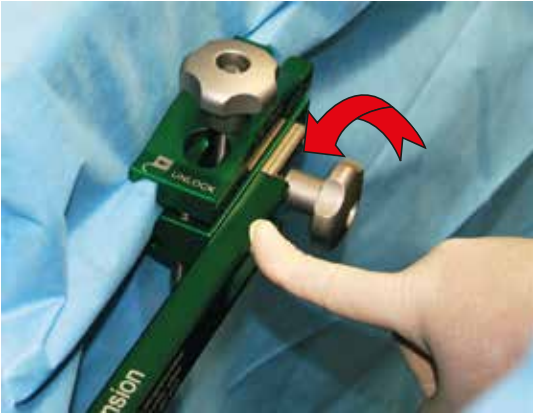

Loosen the side Knob all the way by turning it counter clockwise.

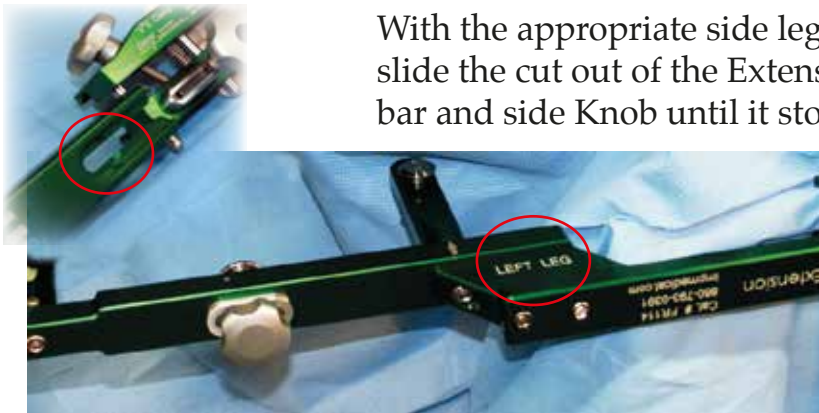


With the appropriate side leg showing on the Extension, slide the cut out of the Extension over the clamp's metal bar and side Knob until it stops.





Turn the Knob clockwise to tighten.

Confirm a minimum of 2 inches of space between the end of the table and Extension.

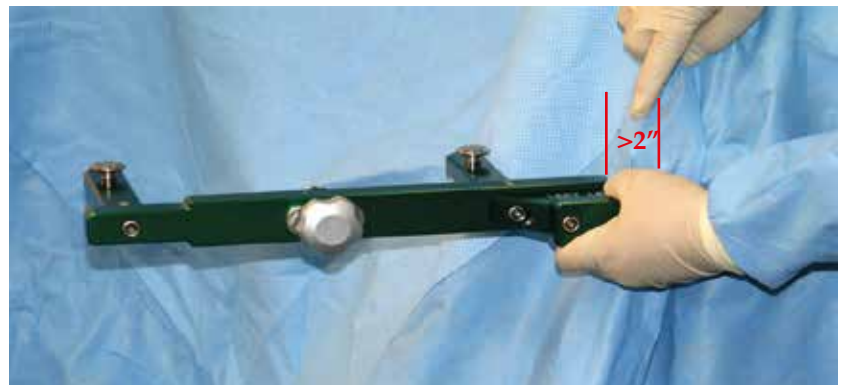


imp De Mayo V² E[®] Knee Positioner IFU ~ **STERILE Extension**


Method 2 : First attach the Extension to the Clamp assuring the side Knob is completely loose by turning it counter clockwise, with the appropriate side leg showing on the Extension. Slide the cut out of the Extension over the clamp's metal bar and side Knob until it stops,

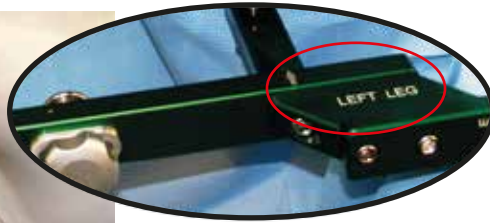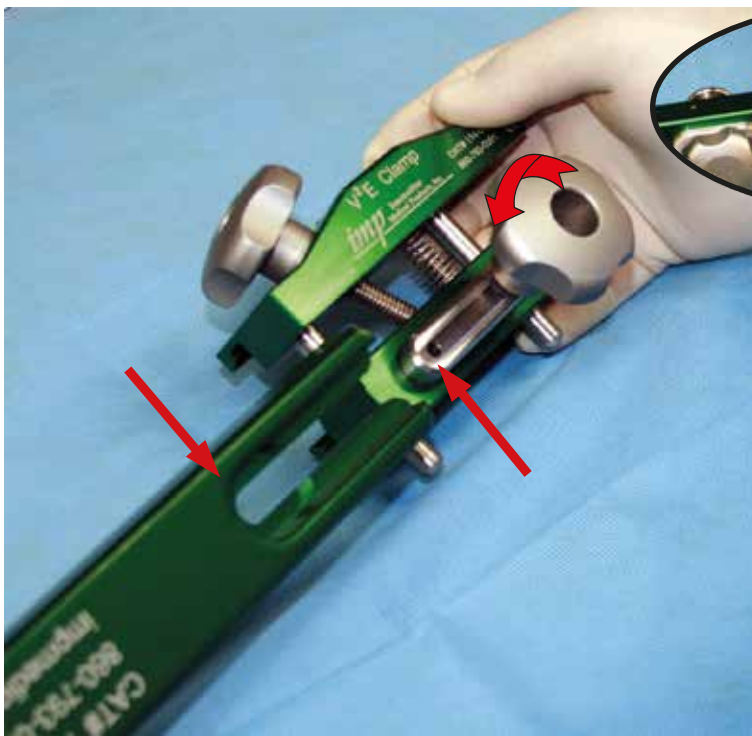

turn the Knob clockwise to tighten.

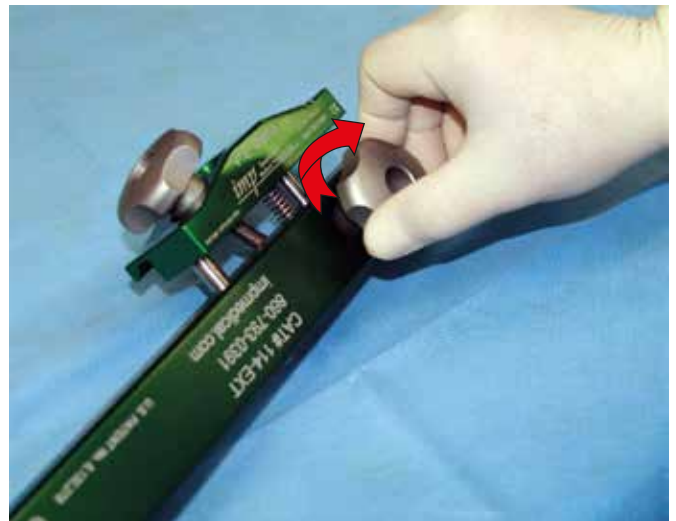


Confirm a minimum of 2 inches of space between the end of the table and Extension.

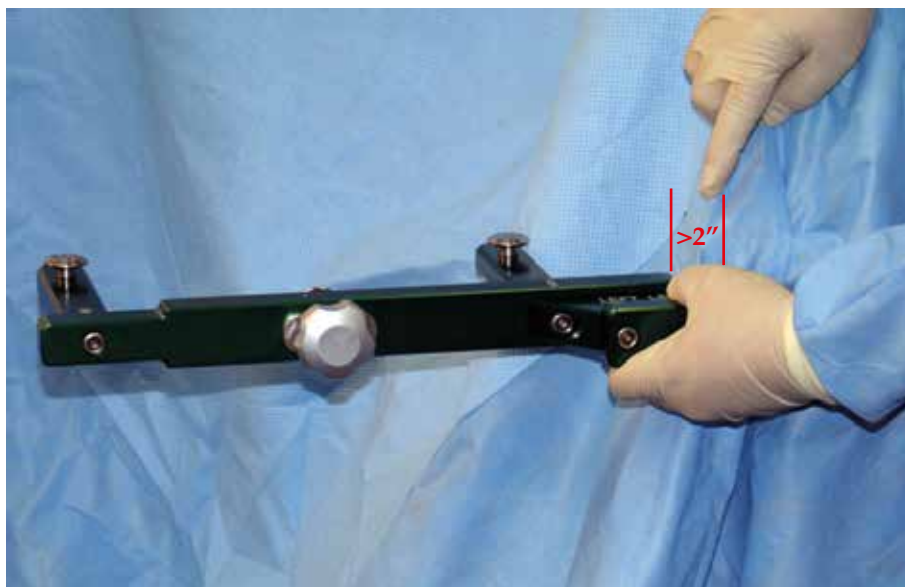

imp De Mayo V² E[®] Knee Positioner IFU ~ STERILE Extension

To lock the clamp to the side rail, squeeze the Jaws and tighten the top Knob.

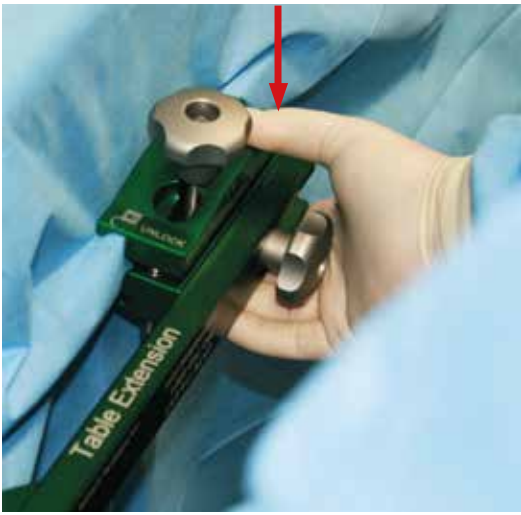


5. Turn the spring loaded Knob counter clockwise all the way in the loose position prior to seating the Base Plate.



To seat the Base Plate on the Extension, line up the white arrows on the Base Plate and Extension.

Pull the Base Plate at an angle against the Receiver Stud and end of the spring loaded Knob.

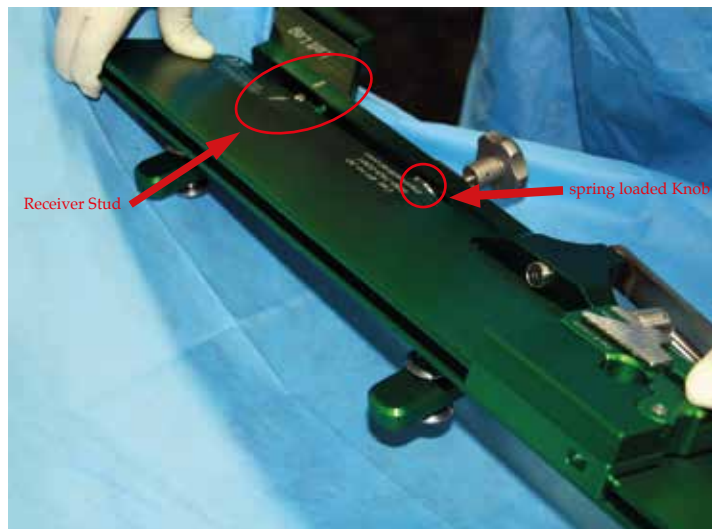



imp De Mayo V² E[®] Knee Positioner IFU ~ **STERILE Extension**

Push down on the Base Plate for final seating, and finger tighten the Knob turning clockwise.



To remove, loosen Knob all the way turning counter clockwise then pull the Base Plate and lift to remove.

6. Protect your patient from potential Pressure Ulcers. Insert the *imp* Patient Protective Pad into the sterile boot.

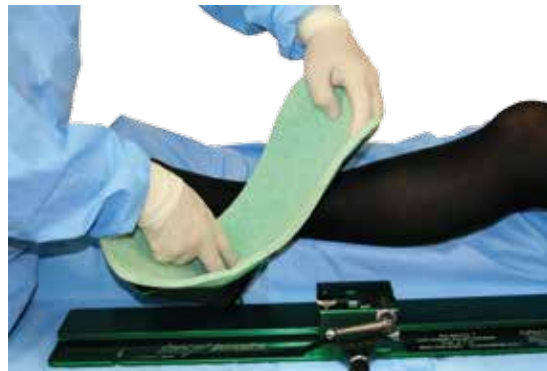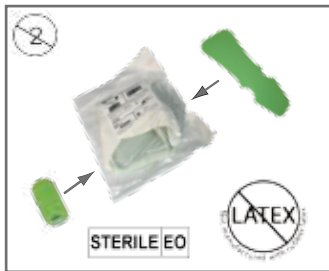

7. To insert the pad, position the carriage under the patient's foot with the knee flexed.



imp De Mayo V² E[®] Knee Positioner IFU ~ **STERILE Extension**



Insert the ball of the boot into the carriage and tighten the E-Brake.

Slide the varus tilt lever to Closed.



For additional holding, rotate the Locking Lever to the right.

8. Place the patient's foot in the boot and wrap cohesive bandage around the foot starting with a minimum of six (6) foot wraps, tear, and finish the wrap above the bracket around the calf.



Avoid wrapping cohesive bandage over the bracket on the back of the boot.




imp De Mayo V² E[®] Knee Positioner IFU ~ **STERILE Extension**

9. To reposition the leg:

Slide the varus tilt lever to Open,



unlock the E-Brake by turning it counter clockwise,

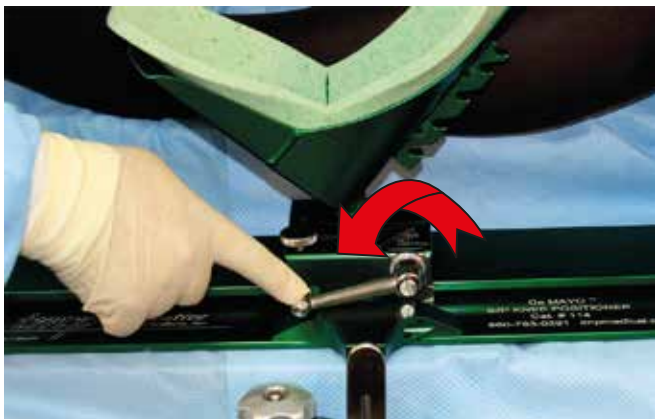

and loosen the Locking Lever by rotating to the left.

10. Rotate the Carriage Locking Lever to the right to lock the boot.




imp De Mayo V² E[®] Knee Positioner IFU ~ **STERILE Extension**

The doctor may approach the knee procedure between the legs of the patient.



Set Up Test Procedure

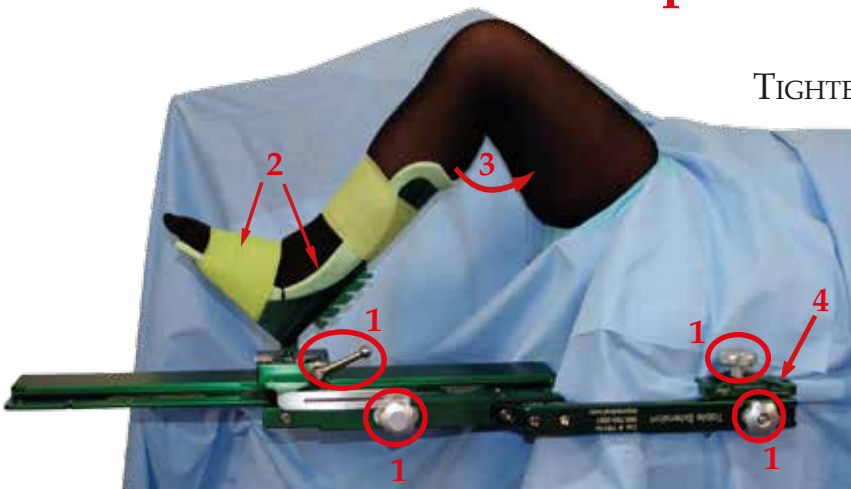

TIGHTEN CLOCKWISE TO LOCK KNOBS AND LEVERS:

- 1 ~ Check all knobs and levers are locked
- 2 ~ Patient is fully protected by *imp*[®] foam and cohesive wrap
- 3 ~ Patient's Leg can reach full flexion
- 4 ~ Sterile Extension Arm is fully seated into V²E[®] Clamp



imp[®] products are protected by patent & patent pending rights ~ go to impmedical.com/patents
© August 2019 Innovative Medical Products, Inc.

ALL RIGHTS RESERVED

imp[®] **Innovative**

Medical Products, Inc.

87 Spring Lane, PO Box 8028

Plainville, CT 06062

PH: (860) 793-0391 FAX: (866) 459-1805

Toll Free PH: 1-800-467-4944

www.impmedical.com

