

Info Sheet

Quad[®] Clamp Disassembly and Reassembly

Catalog Numbers	120	Effective Date	October 6, 2021	Status	APPROVED
Doc ID	INFO-IMP-0002	Version	4	Page 1 of 2	

Disassembly

1. Remove thumb screw



2. Remove the knob completely



3. Pull the T post out of the center of the clamp




4. Separate the top and bottom jaw



Reassembly

1. Slide the bottom jaw into the top jaw

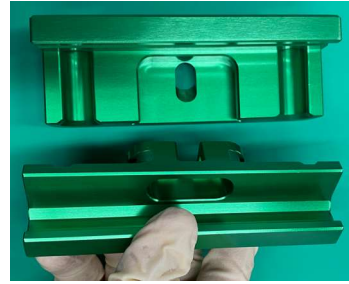

2. Replace the T post by dropping it into the recessed hole



3. Ensure that the threaded rod is protruding from the back of the clamp



4. Replace the knob onto the threaded rod



5. Replace thumb screw



Scan for additional documentation



Quad® Clamp is a registered trademark of Innovative Medical Products, Inc.
 imp® products are protected by patent & patent pending rights ~ go to impmedical.com/patents
 © October 2021 Innovative Medical Products, Inc.
 ALL RIGHTS RESERVED



Innovative Medical Products
 87 Spring Lane
 Plainville, CT 06062
 P: 860-793-0391
 F: 866-459-1805