

Info Sheet

De Mayo Adapt2Fit™ Modular Knee Positioner Carriage Removal and Replacement

Catalog Number	1020	Effective Date	April 20, 2021	Status	APPROVED
Doc ID	INFO-IMP-0005	Version	1	Page 1 of 3	

Carriage Removal

Option 1: Middle Removal


1. With the baseplate split apart, pull the carriage from the baseplate



Remove the handle and E-Break for cleaning

Option 2: End Removal

1. Unscrew the thumbscrew with attached spring and remove the handle, then remove entire threaded rod with attached L Plate using your fingers.

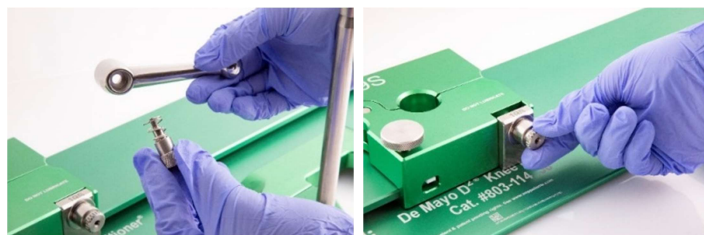

2. Slide the carriage toward the right hand side of the baseplate and slide the carriage around the curved end of the top plate.



3. Remove the E-Break from the carriage.




Carriage Replacement

Option1: Middle Replacement

1. Ensure the Teflon pad is fully seated and continue to hold it while installing the carriage



2. While still holding the Teflon pad slide the carriage onto the left hand baseplate

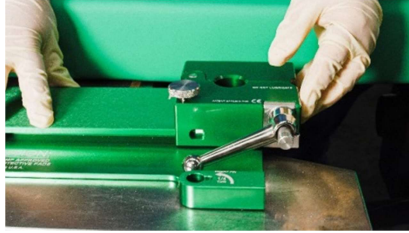

Option 2: End Replacement

1. While holding the Teflon pad, slide the carriage onto the right hand baseplate, the around the curved end of the baseplate.



2. Using your fingers screw in the threaded rod with attached L Plate into the carriage until a boot ball fits into the carriage.



3. Replace the handle in the 3 o'clock position and screw in the thumbscrew with attached spring until tight.




4. With the boot locked into place, pull the handle out and use the thumbscrew to make any final adjustments and replace the E-Brake.



Safety Test

1. Check that the handle locks the boot Ball and is in the 3 o'clock position
2. The E-Break is replaced

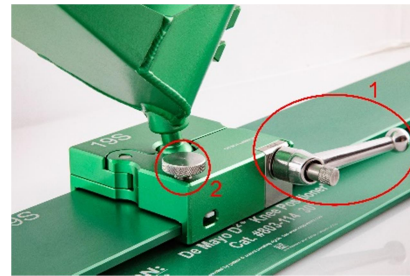

Scan for additional documentation



imp® products are protected by patent & patent pending rights ~ go to www.imppatents.com
 © April 2021 Innovative Medical Products, Inc.
 ALL RIGHTS RESERVED



Innovative Medical Products
 87 Spring Lane
 Plainville, CT 06062
 P: 860-793-0391
 F: 866-459-1805